

CapKeeper® Test Strip Cap & Container Organizer Instructions For Use

K100-1250

fits 1-1/4" caps



Measure outside diameter of your test strip container cap to determine correct CapKeeper® part number for use.

K100-1375 — fits 1-3/8" caps

K100-1750 — fits 1-3/4" caps



Cap cover



Keep cap (upper section)

Discard tear away retainer (lower section) on new strip containers.



K100-1249

fits 1-1/4" caps



Loop



Standalone

Push/pull-off Cap Style

1. Place loop around test strip container.
2. Push capped test strip container completely into Capkeeper™ cap cover.

Push/pull-off Cap Style

1. Push capped test strip container completely into Capkeeper™ cap cover.
2. To open/close strip container: grip and squeeze cap cover while pulling/ pushing on test strip container.

Rotate-on/off Cap Style

1. Push capped test strip container completely into CapKeeper™ cap cover.
2. To open/close strip container: grip and squeeze cap cover while rotating test strip container.

3. To open/close strip container: grip and squeeze cap cover while pulling/ pushing on test strip container.
4. Note: Discard tear away retainer (lower section of cap) on new test strip containers.