

Isopure Corp

Step 2 Cleaner

Handling, Storage and Usage Instructions

Observe all safety precautions prior to handling this product



Face Shield or Safety Goggles



Protective Apron



Chemical Resistant Gloves

Health: 3 (Hazardous)

Fire: 0 (Minimal)

Reactivity: 0 (Minimal)

Intended Use: This product is designed to be used in the MD400 series Reverse Osmosis System or equivalent only. **Never mix this chemical with other chemicals or use this chemical in a manor inconsistent to its intended use. Consult MSDS sheet for further details.**

- 1) Fill HydroBLAST tank.
- 2) Empty the entire contents of 1 (one) bottle into HydroBLAST tank.
- 3) Check pH of tank (target 10-12). Add an additional bottle if <10.
- 4) Allow to recirculate for 7-10 min.
- 5) Add one bag of HydroBLAST enhancer.
- 6) Recirculate for an additional 7-10 min.
- 7) Rinse out.

Handling Precautions: Wear safety glasses, goggles, or face shield, safety gloves, personal protective clothing as required in section 8 in the MSDS.

Storage Requirements: Store in closed container, cool dry, in well-ventilated area away from source of heat, and incompatible materials.

First Aid:

Inhalation: Move to fresh air, give oxygen if difficulty in breathing.

Eye Contact: Flush with water for 15 min, including under eyelids.

Skin Contact: Flush with water for 15 min.

Ingestion: Give water to drink to dilute material. **Induce Vomiting.** In all cases **Get Medical Attention**

Consult MSDS sheet for further details. Available at Isopure.com, products, MD400 Series.

Contact Isopure Corp. at (502) 722-1000 if any questions arise concerning any aspect of this document.