

CalRite® QC Standard Solutions ProbeRite® Sensor Cleaning Solution Thawing Frozen Solution Instructions

CalRite® QC Standard Solutions and ProbeRite® Sensor Cleaning Solutions may freeze in transit during the winter months. Refer to the following instructions if you receive product that is frozen and the container is undamaged.

Thawing Instructions:



- The container may swell and become distorted when frozen. Inspect tamper-resistant seal to be sure it is still intact. If container is damaged or seal is not intact and/or solution is leaking, do not use.
- Gradually warm the frozen product to room temperature until completely thawed.
- Invert the product several times to remix any potentially separated or stratified layers which will dissolve any crystallized material. Vigorous agitation may be necessary.
- Once the solution is properly thawed and mixed, inspect for cloudiness or any undissolved material.
- Do not use if solution remains cloudy or undissolved particles are visible.
- Refer to the product 'Instructions for Use' once product is properly thawed and remixed.

RPC • 1643 W Modern Court • Tucson, AZ • USA • (520) 888-5551